Cleans and sanitizes CPAP
mask and accessories

Lumin[®]

USER MANUAL

 **3B[™] Medical, Inc.**

www.3BLumin.com

Cleans and sanitizes CPAP mask and accessories

Lumin[®]

USER MANUAL

 **3B[™] Medical, Inc.**
www.3BLumin.com

ENGLISH

EN

Indications for Use	5
Specifications	6
Understanding your Lumin cleaner	7
Using your Lumin RACK	7
Using your Lumin	8
Maintenance of your Lumin	9

EN	User manual	2
DE	Benutzerhandbuch	10
ES	Manual de usuario	18
FR	Manuel de l'Utilisateur	26
NL	Handleiding	34
DK	Brugermanual	42
IT	Manuale d'uso	50

Thank you for your recent purchase of the Lumin CPAP Mask and Accessories Sanitizer. The Lumin was developed by 3B Medical, a manufacturer and distributor of CPAP devices and masks.

The Lumin device may be used with all brands of masks and accessories.

The Lumin disinfects using a very specialized and high powered UVC light source, the same type of light source used in hospital operating rooms and sterile clean rooms. UVC light will disinfect up to 99% of harmful bacteria, pathogens and fungi that can cause infection and illness. UV light is also the safest disinfection option on the market, there is NO HARMFUL OZONE. Ozone has been shown by the EPA to irritate the lungs and the respiratory system, making it extra harmful for people with respiratory illness.

The 3B Lumin is intended to supplement your CPAP accessory manufacturers cleaning recommendations.

Please read and follow all directions in this user manual.

Indications for Use

The Lumin is a device intended to disinfect Continuous Positive Airway Pressure (CPAP) accessories, including mask and water chamber. A person using a PAP device to treat Obstructive Sleep Apnea (OSA) may use Lumin to sanitize their mask and water chamber to augment their PAP equipment manufacturer's recommended cleaning instructions. The Lumin is intended for home use.

Warning - the Lumin is not a toy and is not intended to be used by children.

Warning - the Lumin has a powerful UV light bulb included, use with care in case of broken glass.

Warning - do not use in or around sinks, tubs, pools or any bodies of water.

Warning - the UV that is emitted in the closed chamber is not compatible with life, never place living creatures in the Lumin.

Warning - do not stack objects on top of your Lumin.

Warning - use this device only for its intended use as described in this manual. If the equipment is not used in the manner specified, the protection provided by the equipment may be impaired.

Warning - Do not attempt to operate this device with the drawer open. The device was designed to shut off automatically when the drawer is open. It is important not to defeat or tamper with this safety feature as UVC light can cause damage to eyesight.

Warning - The Lumin plugs into an outlet. Disconnect the device by unplugging. Do not plug the device into an outlet that is difficult to access.

SPECIFICATIONS

SKU:	LM3000EU
Weight:	5.5 lbs
Dimensions:	12.25 x 8.5 x 7.75"
Voltage:	AC 110 V 60Hz AC 220 V 50Hz
Power Rating:	13W
Environmental Conditions:	
	Storage Temperature: 0° to 120° F / -18° to 49° C
	Operating Temperature: 20° to 100° F / 7° to 38° C
	Humidity: Up to 93%, non-condensing

EXPLANATION OF SYMBOLS USED

Equipment protected throughout by double insulation or reinforced insulation.

Refer to instruction manual.

Important information concerning your safety or the operation of your device.

Understanding Your Lumin Cleaner

The Lumin operates using a specialized high powered germicidal UV bulb. The drawer is made of polished aluminum which reflects the UV light to improve coverage. UV disinfection is effective on bacteria, viruses, mold and fungi and is well suited for keeping your CPAP mask and humidifier water chamber free from bacteria that can make you sick. CPAP masks and the humidifier are in contact with warm moist air throughout the night, the ideal environment for breeding bacteria. Daily use of the Lumin will help keep you healthy and compliant with your CPAP equipment.

Because the Lumin operates using a form of high energy light, it is very effective in disinfecting your CPAP accessories. To keep your mask, hose and water chamber clean and reduce the risk of infections, we recommend using Lumin twice weekly on your mask, water chamber and hose. Disassembly of your mask is not required for routine disinfection. If visible debris is present remember to wash or use a CPAP wipe on the mask prior to disinfection. A disassembled mask can be placed in the Lumin for an even more thorough disinfection. Hoses do benefit from Lumin disinfection, and the minimum recommendation of twice weekly will prevent build-up of biofilm in the hard to reach areas. Maintain healthy breathing with quarterly replacement of your CPAP hose.

Using Your Lumin RACK

Use of the clear Lumin RACK is suggested to improve disinfection. Set your mask and accessories on the RACK to allow greater penetration of the underside of the article being sanitized. If your item is large, use of the Lumin RACK is optional and it may be removed.

Using your Lumin

1. Plug the Lumin power cord in a wall outlet.
2. For your humidifier water chamber, open the water chamber and lay both sides facing up. For your CPAP mask lay the mask with the inside facing up and laying flat for maximum UV penetration.
3. If your mask or accessories are visibly soiled please first wipe thoroughly with a damp cloth or a CPAP wipe.
4. Close the Lumin chamber drawer.
5. Press round power button, the Cycle-On orange light will turn on indicating that the 5 minute disinfection cycle has started.
6. A light glow of UV Light will be visible through the tinted oval window on the top of the unit while disinfection is occurring.
7. After 5 minutes the Lumin will emit a beep and the UV light will no longer be visible. A green light will indicate completion of the cleaning cycle. You may now remove your equipment. No waiting period is necessary and your mask and equipment are now ready to use.

Caring for your Lumin Device

The Lumin bulb is designed to exceed the life of your device so changing the bulb should not be necessary. If, however, it does become necessary, the bulb is removed by simply grabbing the base and pulling firmly. The bulb is easily replaced by inserting the new bulb and pushing it into the socket.

The Lumin drawer is lined with polished aluminum. Do not use baking soda or other alkali-based cleaners on aluminum as it can cause discoloration. Aluminum scratches fairly easily, so avoid abrasive cleaners and scrubbers when cleaning and polishing. If using a metal or glass cleaner, test the cleaner on a small portion of the drawer. Polish with a soft lint free cloth.

Should the casing of your 3B Lumin become soiled you may wipe down with any moist cloth. Do not immerse your Lumin into water or any cleaning solvents. Place your Lumin on a level counter or tabletop. Do not drop your Lumin or the bulb and or glass drawer may shatter.

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